# Control Syringes

**ARGON MEDICAL DEVICES, INC.**  
 1445 Flat Creek Road  
 Athens, Texas 75751 USA  
 Tel: 800-927-4669  
 Tel: +1 903-675-9321  
[www.argonmedical.com](http://www.argonmedical.com)

**EMERGO EUROPE**  
 Prinsessegracht 20  
 2514 AP The Hague  
 The Netherlands  
 +31 70 345 8570

The symbols glossary is located electronically at [www.argonmedical.com/symbols](http://www.argonmedical.com/symbols)

20-2113-01/0519B

en - English

## Intended Use

These syringes are used in angiographic or radiological procedures where they are used for injection of contrast media into the patient's vasculature to detect blockages or occluded arteries. Some procedures that are commonly performed using the control syringes include: coronary artery studies, ventricular studies, right heart studies, arteriograms, venograms, PTCA studies, PTA studies, and other special radiological studies.


## Contraindications for Use

There are no medical contraindications associated with the syringes. Assessment of patient overall physical state by the physician is required prior to use of the device.

## Warning

- Syringes are intended for use by trained medical personnel in a clinical setting.
- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury.
- Do not use if package appear open or if the expiry date has been exceeded.

## Directions for Usage

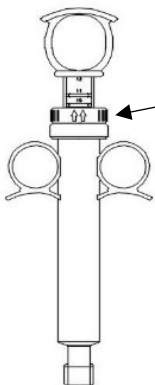- 
- Pull back the plunger until the barrel is filled with contrast medium.
  - Press bilaterally on the ridged sides of the locking mechanism and slide the locking mechanism until the arrows point upward to "0" ccs.
  - After locking in "0" ccs, press bilaterally on the arrows to confirm that the locking mechanism is placed properly.
  - Push the plunger down until resistance is felt.
  - Press bilaterally on the ridged sides of the locking mechanism and slide the locking mechanism to the desired volume of ccs you wish to inject. (Note: the arrows point upward to the volume to be injected.)
  - After locking in the desired dosage, press bilaterally on the arrows to confirm that the locking mechanism is placed properly.
  - Screw on or attach Syringe to delivery system.
  - Push the plunger down until resistance is felt.
  - Discard per facility policy.

**CAUTION: EXCESSIVE FORCE MAY DAMAGE PRODUCT**

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