NeedleVISE®

ATRION MEDICAL PRODUCTS, INC.
1426 Curt Francis Road
PO Box 564
Arab, Alabama, 35016, U.S.A.
PH : 256-586-1580/800-343-9334

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

ENGLISH

Ation NeedleVISE® Large-Bore Sharps Securing Device

Description:

The Atrion NeedleVISE® Large-Bore Sharps Securing Device is a single needle, single patient procedure use, disposable device used for securing contaminated needles. The NeedleVISE® Large-Bore Sharps Securing Device has an internal one-way metal clip for retaining the used 8-27 Gauge (Ga) sharp.

Indications:

The NeedleVISE® Large-Bore Sharps Securing Device is intended for use in various clinical procedures as a sharps securing device for contaminated needles when an integral sharps safety system is not available.

Directions for Use:

Figure 1

1. Place the NeedleVISE® Large-Bore Sharps Securing Device on a flat stable surface. (Figure 1).

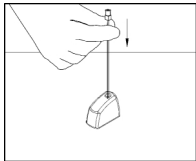
Figure 2

2. Immediately following use of a needle and while still holding it with one hand away from the sharp end, firmly insert the sharp pointed tip straight down into the opening in the NeedleVISE® Large-Bore Sharps Securing Device until it stops, making sure to **KEEP YOUR FREE HAND AWAY FROM THE SHARPS SECURING DEVICE DURING INSERTION**. This action will lock the needle firmly into the sharps securing device. (Figure 2).

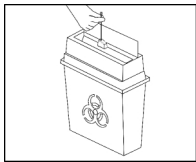
Figure 3

3. Dispose of used NeedleVISE® Large-Bore Sharps Securing Device with secured sharps in an approved sharps container. (Figure 3).

Precautions:

- For single patient procedure use only.
- For single needle use only.
- Never grasp the needle at the sharp end.
- Do not hold NeedleVISE® Sharps Securing Device with free hand while inserting a needle.
- Always use one-handed technique when inserting sharps into NeedleVISE® Large-Bore Sharps Securing Device.
- Do not attempt to remove sharps once secured in NeedleVISE® Large-Bore Sharps Securing Device.
- According to OSHA regulations contaminated needles or other contaminated sharps shall not be bent, recapped or removed unless the employer can demonstrate that no alternative is feasible or such action is required by a specific medical or dental procedure.
- Dispose in a sharps container immediately after the clinical procedure is completed

Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician

For single use only. Do not attempt to reuse.

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