ARGON

MEDICAL DEVICES

ARGON MEDICAL DEVICES, INC.
 1445 Flat Creek Road Athens, Texas 75751 USA
 Tel: +1 (903) 675-9321
 Tel: +1 (800) 927-4669
 www.argonmedical.com


EMERGO EUROPE
 Prinsessegracht 20
 2514 AP The Hague
 The Netherlands
 +31 70 345 8570

Argon Medical Devices UK Ltd
 Eastgate Business Centre
 Eastern Avenue
 Burton-on-Trent
 Staffordshire
 DE13 0AT

IFU9015/0321A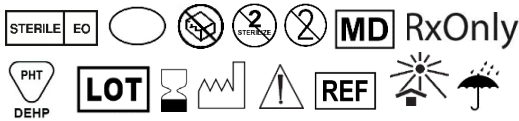

 <https://www.argonmedical.com/resources/product-information>

English	The symbols glossary is located electronically at www.argonmedical.com/symbols
Bulgarian	Речникът на символите е наличен в електронен вид на адрес www.argonmedical.com/symbols
Czech	Slovníček symbolů je elektronicky umístěn na www.argonmedical.com/symbols
Danish	Symbolforklaringen findes elektronisk på www.argonmedical.com/symbols
German	Ein elektronisches Symbolglossar finden Sie unter www.argonmedical.com/symbols
Greek	Το γλωσσάρι των συμβόλων παρέχεται ηλεκτρονικά στη διεύθυνση www.argonmedical.com/symbols
Spanish	El glosario de símbolos puede consultarse electrónicamente en www.argonmedical.com/symbols
Estonian	Sümbolite seletuste kogu on elektrooniliselt saadaval aadressil www.argonmedical.com/symbols
Finnish	Elektroninen symbolisanasto on osoitteessa www.argonmedical.com/symbols .
French	Le glossaire des symboles est disponible au format électronique à l'adresse www.argonmedical.com/symbols
Croatian	Pojmovnik simbola nalazi se u elektroničkom obliku na www.argonmedical.com/symbols
Hungarian	A szimbólumok összefoglalása elektronikusan itt található: www.argonmedical.com/symbols
Italian	Il glossario dei simboli è disponibile in formato elettronico all'indirizzo www.argonmedical.com/symbols
Lithuanian	Simbolių žodyno elektroninė forma pateikta www.argonmedical.com/symbols
Latvian	Simbolu glosārijs ir atrodams elektroniskā veidā vietnē www.argonmedical.com/symbols
Dutch	Uitleg over de symbolen is elektronisch beschikbaar op www.argonmedical.com/symbols
Norwegian	Symbolforklaringen befinner seg elektronisk på www.argonmedical.com/symbols
Polish	Glosariusz symboli w postaci elektronicznej dostępny jest pod adresem www.argonmedical.com/symbols
Portuguese	O glossário de símbolos está localizado eletronicamente no endereço www.argonmedical.com/symbols
Romanian	Glosarul simbolurilor este stocat electronic la www.argonmedical.com/symbols
Slovak	Glosár symbolov je v elektronickej podobe k dispozícii na stránkach www.argonmedical.com/symbols
Slovenian	Slovar simbolov se nahaja v elektronski obliki na www.argonmedical.com/symbols .
Swedish	Symbolforklaringen hittas elektroniskt på www.argonmedical.com/symbols
Turkish	Semboller sözlüğü elektronik ortamda www.argonmedical.com/symbols adresinde yer almaktadır

EN - ENGLISH

Connectors/Stopcock

Intended Use/Purpose: The product are intended for used in conjunction with drainage bags and channeling catheter for fluids.

Devices Description: This product are stand-alone components built to user specifications. The device offers wide range of connecting tube lengths, various types of adapters and stopcocks for all types of drainage bags.

Indication for Use: Connectors are used for channeling blood, urine and other body fluid between catheters and all types of drainage bags.

Warnings

- Contents are supplied sterile and are intended for single use only. Do not re-sterilize, reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury.
- Inspect the package integrity before use.
- Do not use if package is open or damaged and if the expiry date has been exceeded.
- Do not continue to use if any of the component are damaged during the procedure.

Precautions

- This device is intended for use by trained, qualified licensed physician.

Direction for Use:

- Identify the male and/or female ends of connectors.
- Insert male and female luer locking ends together.
- Twist to confirm secure connection.

Possible Complications:

There is no known complication related to this device.

Storage: Store at controlled room temperature.

Disposal: After use, this product may be a potential biohazard. Dispose in accordance with applicable laws and regulations.

NOTE: In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at quality.regulatory@argonmedical.com as well as to the competent health authority where the user/patient resides.

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